

Pictello with 2 Switch Control

Introduction

This tutorial explains how to set up Pictello with 2 switches using the Pretorian APPLICator Bluetooth switch interface.

Pictello is an app that has been designed to work with 1 or 2 switches in addition to touch screen control.

This tutorial requires:

- 2 wired switches
- Pretorian APPLICator switch interface
- iPad iOS version 15.6.1 or higher
- Pictello app - Assistiveware

Charging the APPLICATOR

1

Charge the APPLICATOR using the cable provided:

Plug the cable into a USB Port.

The charging light will be green.

The light will disappear once the device is fully charged.




Configure the APPLICATOR

2

Wake up the APPLICATOR by pressing any button.

A swirling pattern will appear in the display area.

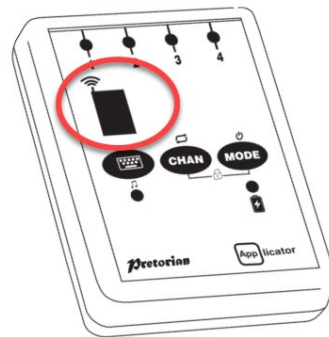


- Pair the APPLICATOR to the iPad.
- 3** Open the **Settings** menu.

Slide **Bluetooth** to **On**.

The APPLICATOR will appear as a discoverable device.

Select the **APPLICATOR** to complete pairing.


The swirling light in the display area will disappear. Confirm that the device is **Connected**.

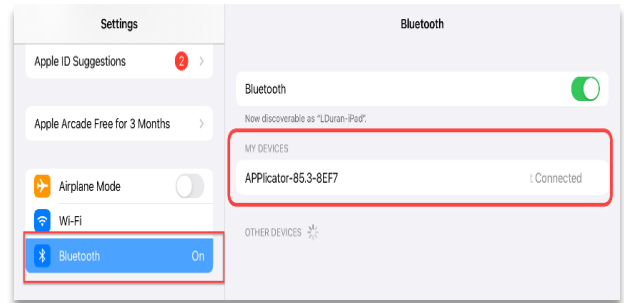

Program APPLICATOR Channel 1 to Mode 5:

- 4** Connect a wired switch to **Channel 1**.
- Press the **Channel** button to move the light to **Channel 1**.


Press the **Mode** button on the APPLICATOR to 5 (Space).

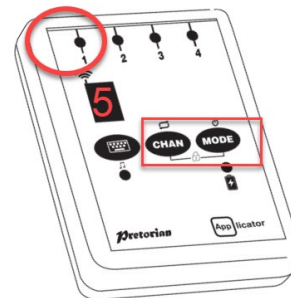

Program APPLICATOR Channel 2 to Mode 6:


- 5** Connect a wired switch to **Channel 2**.
- Press the **Channel** button to move the light to **Channel 2**.

Press the **Mode** button on the APPLICATOR to 6 (Enter).

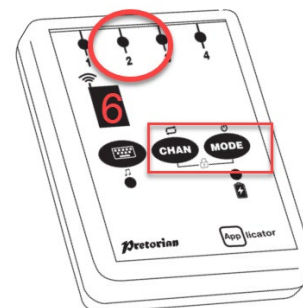


Configure Pictello 2 Switch Access

- 6
 - Launch Pictello
 - Tap **Settings**.
 - Choose **Configure Switches**.

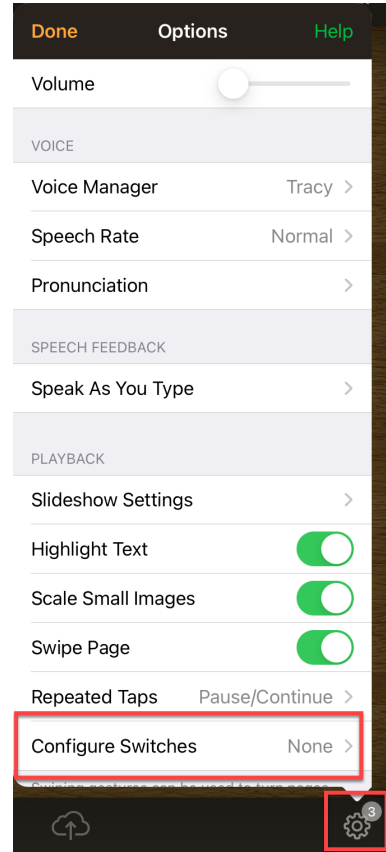


- 7
 - Choose the 2 **Switches** option.
 - Tap the **i** icon.

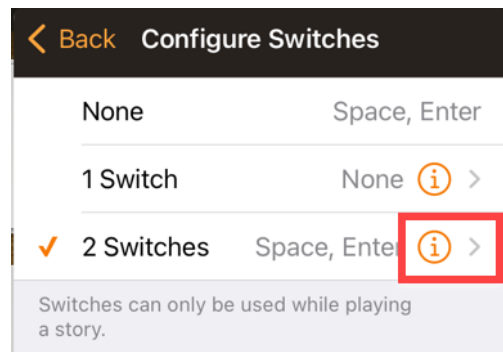

- 8 Tap **Read page**.
Choose **Space**.

Press **Back**


Tap **Next page**.
Choose **Enter**.
Close this menu.

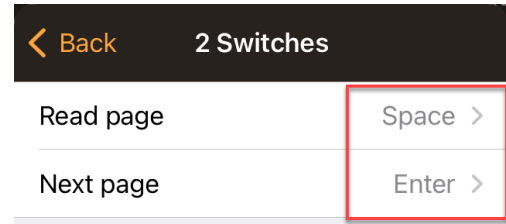

You now have the following configuration:

Switch 1 = Read page
Switch 2 = Next page

- 9 Open the Pictello book you wish to read.
Select **Switch** on the home page.

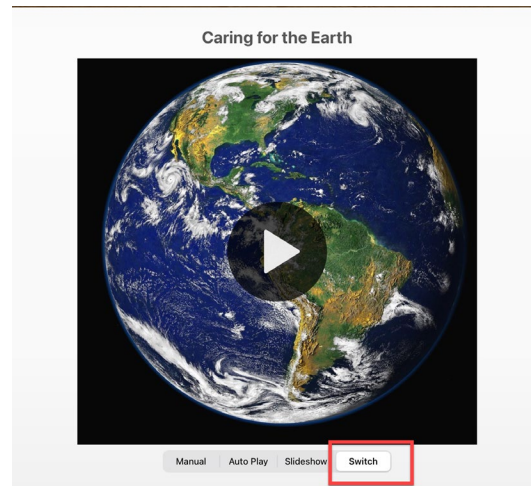

- 10 Note that the cover is NOT switch accessible.
Tap the screen to start the story
Press the switches to read the story and turn the pages.

