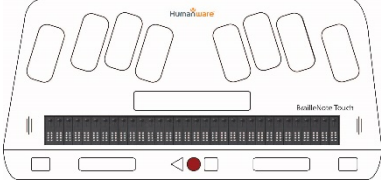
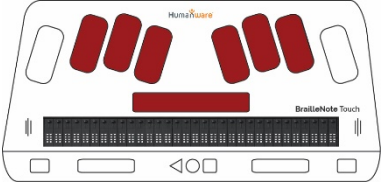
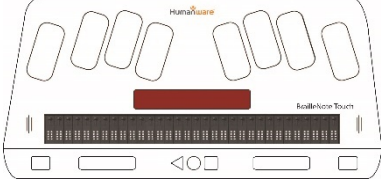
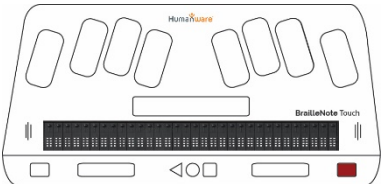
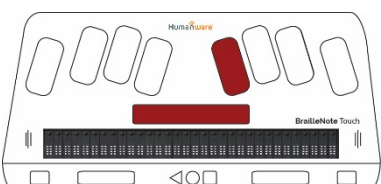
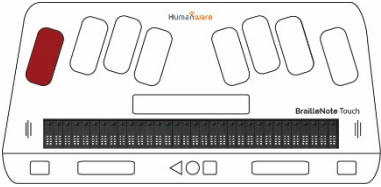
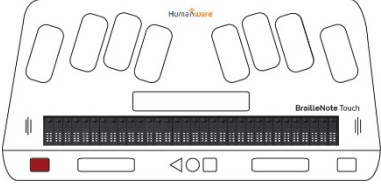
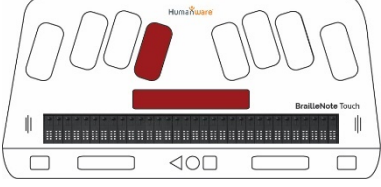
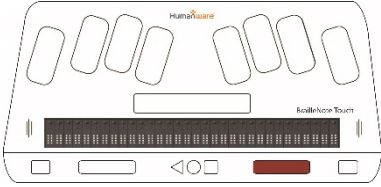
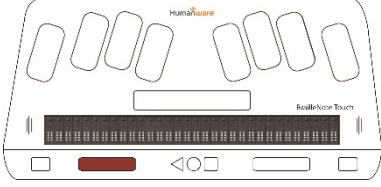
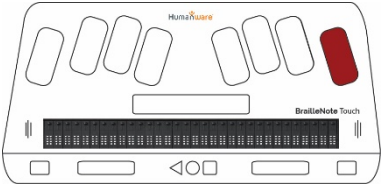
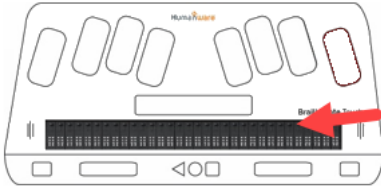
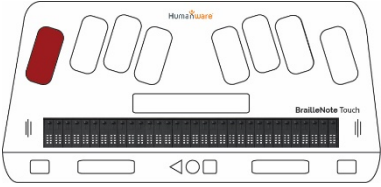
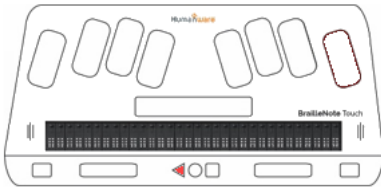
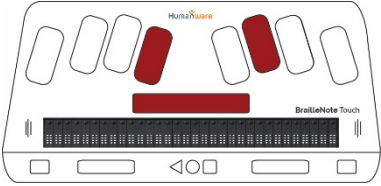
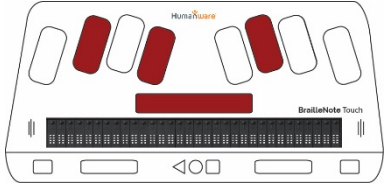
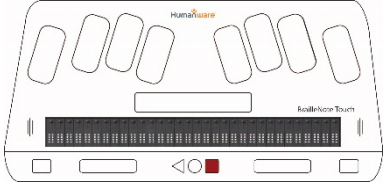
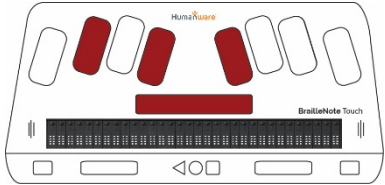
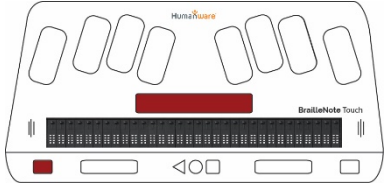







1.0 BrailleNote Touch Plus – Basic Keyboard Commands






<p style="text-align: center; font-size: 2em;">1</p>	<p>MAIN MENU:</p> <p>Press the HOME button</p> <p style="text-align: center;">OR</p> <p>SPACEBAR+DOT 1-2-3-4-5-6</p>	 <p style="text-align: center;">HOME</p>  <p style="text-align: center;">SPACEBAR+DOT 1-2-3-4-5-6</p>
<p style="text-align: center; font-size: 2em;">2</p>	<p>NEXT ITEM:</p> <p>To move to the next item on a menu press the SPACEBAR</p> <p style="text-align: center;">OR</p> <p>NEXT button</p> <p style="text-align: center;">OR</p> <p>SPACEBAR+DOT 4.</p>	 <p style="text-align: center;">SPACEBAR or</p>  <p style="text-align: center;">NEXT or</p>  <p style="text-align: center;">SPACEBAR+DOT 4</p>

<p style="text-align: center; font-size: 2em;">3</p>	<p>PREVIOUS ITEM:</p> <p>To move to the previous item on menu press the BACKSPACE (7)</p> <p style="text-align: center;">OR</p> <p>PREVIOUS button</p> <p style="text-align: center;">OR</p> <p>SPACEBAR+DOT 1.</p>	 <p style="text-align: center;">BACKSPACE or</p>  <p style="text-align: center;">PREVIOUS or</p>  <p style="text-align: center;">SPACEBAR+DOT 1</p>
<p style="text-align: center; font-size: 2em;">4</p>	<p>PANNING BRAILLE ON REFRESHABLE DISPLAY:</p> <p>To pan braille to the right, press the RIGHT THUMB KEY.</p> <p>To pan braille to the left, press the LEFT THUMB KEY.</p>	 <p style="text-align: center;">RIGHT THUMB KEY</p>  <p style="text-align: center;">LEFT THUMB KEY</p>

<p>5</p>	<p>ENTER: Opens or activates the selected item.</p> <p>Press DOT 8 /ENTER</p> <p>OR</p> <p>any CURSOR ROUTING KEY.</p>	 <p>DOT 8</p>  <p>CURSOR ROUTING KEYS</p>
<p>6</p>	<p>DELETE TEXT:</p> <p>Press DOT 7, also known as BACKSPACE, to delete text positioned to the left of the cursor.</p>	 <p>DOT 7 / BACKSPACE</p>
<p>7</p>	<p>EXIT MENU OR APPLICATION:</p> <p>To exit a menu or application, press the triangular BACK button</p> <p>OR</p> <p>SPACEBAR+E</p>	 <p>BACK</p>  <p>SPACEBAR+E</p>

8	<p>1st LETTER NAVIGATION:</p> <p>Pressing the FIRST LETTER in the name of an item in a menu or file explorer will move the cursor focus to the next item beginning with that letter.</p>	<p>Type the letter on the traditional keyboard or use TouchBraille directly on the screen to use 1st letter navigation.</p>
9	<p>OPTIONS MENU:</p> <p>To open the KeySoft options menu, press SPACEBAR+O.</p>	 <p>SPACEBAR+O</p>
10	<p>CONTEXT MENU:</p> <p>To open a context menu, press and hold the RECENT APPS BUTTON (also known as CONTEXT MENU)</p> <p>OR</p> <p>Press SPACEBAR+M</p>	 <p>RECENT APPS / CONTEXT MENU</p>  <p>SPACEBAR+M</p>
11	<p>TURN SPEECH ON/OFF:</p> <p>To toggle speech on and off, press SPACEBAR+PREVIOUS.</p>	 <p>SPACEBAR+PREVIOUS</p>
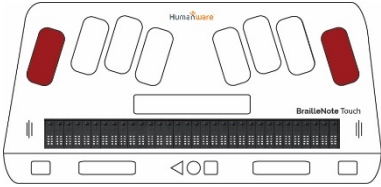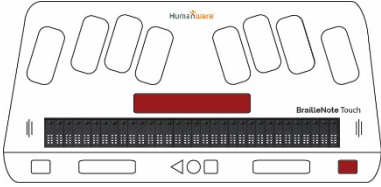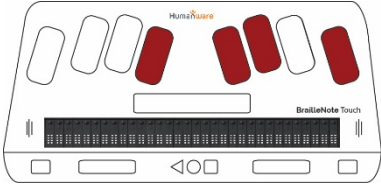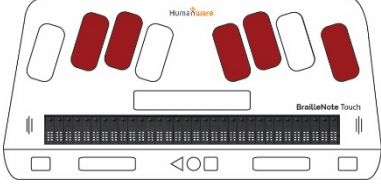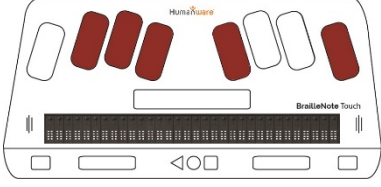
<p>12</p>	<p>STOP SPEECH WHILE SPEAKING:</p> <p>To stop the speech while device is speaking, press BACKSPACE+ENTER.</p>	 <p>BACKSPACE+ENTER</p>
<p>13</p>	<p>TURN VISUAL DISPLAY ON/OFF:</p> <p>To toggle visual display on and off, press SPACEBAR+NEXT.</p>	 <p>SPACEBAR+NEXT</p>
<p>14</p>	<p>CURRENT DATE:</p> <p>To announce the current date, press ENTER+D.</p>	 <p>ENTER+D</p>
<p>15</p>	<p>CURRENT TIME:</p> <p>To announce the current time, press ENTER+T.</p>	 <p>ENTER+T</p>
<p>16</p>	<p>BATTERY STATUS:</p> <p>To announce battery level, press ENTER+P.</p>	 <p>ENTER+P</p>
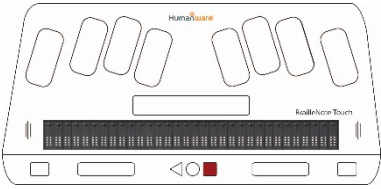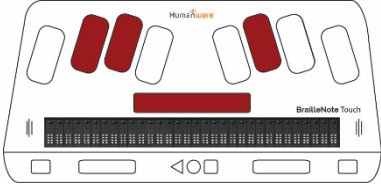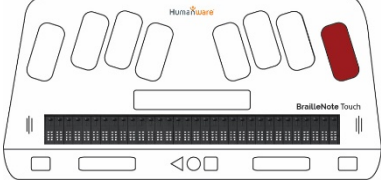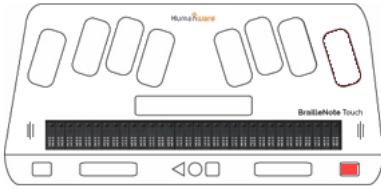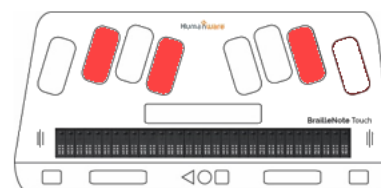
<p>17</p>	<p>SHOW ALL CURRENTLY RUNNING APPS:</p> <p>Press the RECENT APPS button</p> <p>OR</p> <p>SPACEBAR+DOT2-3-5</p> <p>TO CLOSE ONE APP:</p> <p>Move forward through the list of recent apps and stop on the Dismiss ___ button on the app you want to close. Press ENTER or a CURSOR ROUTING KEY.</p> <p>TO CLOSE ALL RUNNING APPS:</p> <p>Stop on the CLEAR ALL button. Press ENTER or a cursor routing key.</p>	 <p>RECENT APPS</p>  <p>SPACEBAR+DOTS2-3-5</p>  <p>ENTER</p>
<p>18</p>	<p>UNLOCK THE DEVICE AFTER TURNING ON OR WAKING:</p> <p>Press the NEXT thumb key until you hear “unlock,” and then press ENTER or a CURSOR ROUTING KEY</p> <p>OR</p> <p>Use first-letter navigation “U”</p>	 <p>NEXT</p>  <p>DOTS 1+3+6</p>