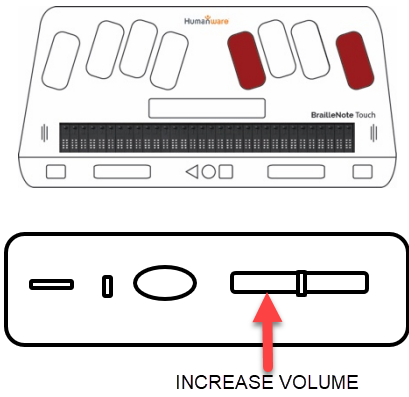
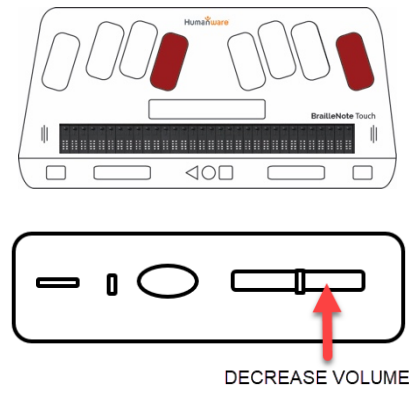
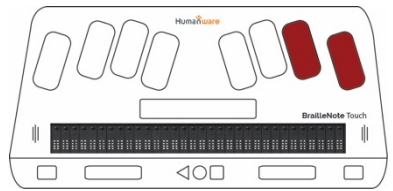
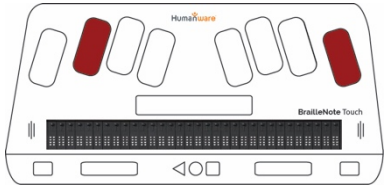
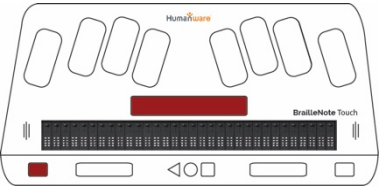
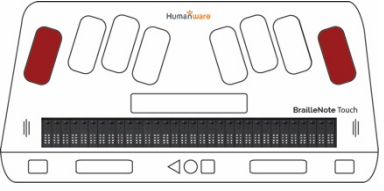



1.2 BrailleNote Touch Plus: Adjusting Speech Settings

Volume		
1	<p>LOUDER To raise the accessibility volume, press ENTER + Dot 4.</p> <p>The increase volume button located on the left face of the device will sometimes increase <i>ring volume</i> and sometimes accessibility volume, depending on context.</p>	
2	<p>SOFTER To lower the volume, press ENTER + Dot 1.</p> <p>The decrease volume button located on the left face of the device will sometimes decrease <i>ring volume</i>, depending on context.</p>	

Reading Rate		
1	<p>FASTER To increase the reading rate, press ENTER + Dot 6.</p>	

<p>2</p>	<p>SLOWER To decrease the reading rate, press ENTER + Dot 3.</p>	
----------	--	---

<h2>Adjusting Speech Settings</h2>		
<p>1</p>	<p>TOGGLE SPEECH ON/OFF To turn speech on or off, press SPACEBAR + Previous. You will need to turn speech back on to hear highlighted areas as you navigate.</p>	
<p>2</p>	<p>PAUSE SPEECH To pause speech while device is speaking, press BACKSPACE + ENTER. If you navigate to a new highlighted area, you will hear speech again.</p>	
<p>3</p>	<p>START SPEECH To start speech again press SPACEBAR + G.</p>	
<p>4</p>	<p>REPEAT To have device repeat what was last spoken, press SPACEBAR + R.</p>	