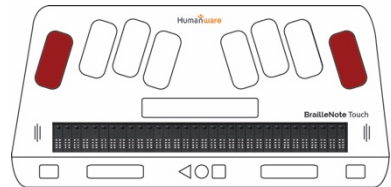
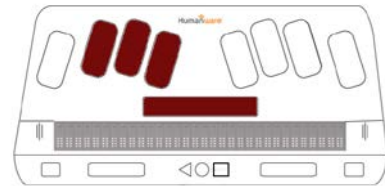
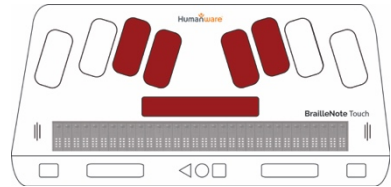


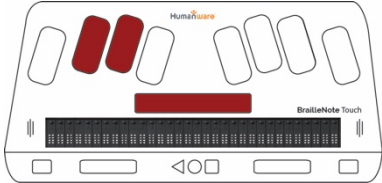
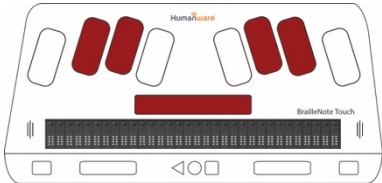
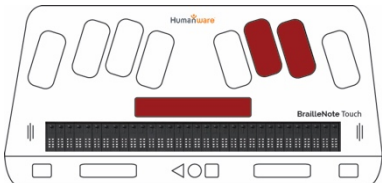
BrailleNote Touch Plus 2.4 – Reading Documents

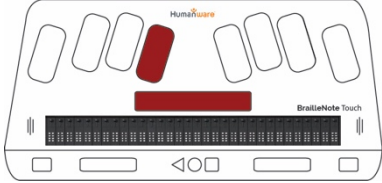

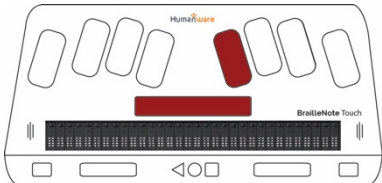
Word Processor (KeyWord): Reading a Document	
	<p>READ ALL To read the entire document from where the cursor is, press SPACEBAR + G (Dot 1245) (G stands for “Go”).</p> <p>To navigate to the top of the document and begin reading there, first press SPACEBAR + Dot 123.</p>
	<p>PAUSE READING To pause reading, press BACKSPACE + ENTER. Press SPACEBAR + G to resume reading from the spot where you paused.</p>

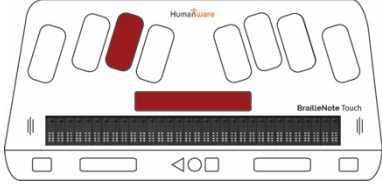
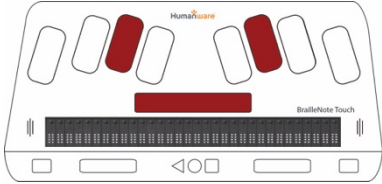
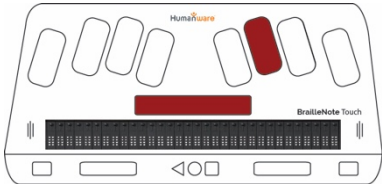


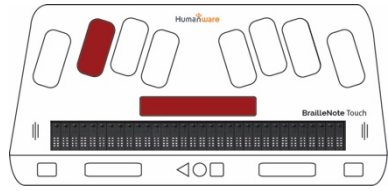
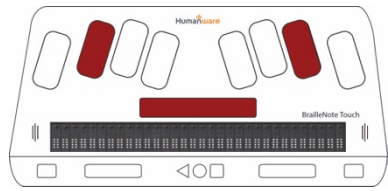
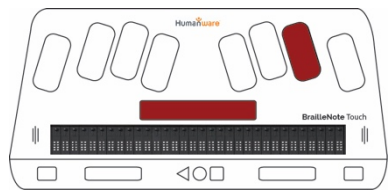
BrailleNote Touch Plus

Word Processor (KeyWord): Reading by Paragraph

	<p>READ PREVIOUS PARAGRAPH To read the previous paragraph, press SPACEBAR + Dot 23.</p>	
	<p>READ CURRENT PARAGRAPH To read the current paragraph, press SPACEBAR + Dot 2356.</p>	
	<p>READ NEXT PARAGRAPH To read the next paragraph, press SPACEBAR + Dot 56.</p>	

Word Processor (KeyWord): Reading by Line	
<p>PREVIOUS LINE To read the previous line, press SPACEBAR + Dot 1.</p>	
<p>CURRENT LINE To read the current line, press SPACEBAR + Dot 14.</p>	
<p>NEXT LINE To read the next line, press SPACEBAR + Dot 4.</p>	

Word Processor (KeyWord): Reading by Word		
	<p>PREVIOUS WORD To read the previous word, press SPACEBAR + Dot 2.</p>	
	<p>CURRENT WORD To read the current word, press SPACEBAR + Dot 25.</p>	
	<p>NEXT WORD To read the next word, press SPACEBAR + Dot 5.</p>	

Word Processor (Keyword): Reading by Character	
<p>PREVIOUS CHARACTER To read the previous character, press SPACEBAR + Dot 3.</p>	
<p>CURRENT CHARACTER To read the current character, press SPACEBAR + Dot 36.</p>	
<p>NEXT CHARACTER To read the next character, press SPACEBAR + Dot 6.</p>	

Tip...

Remember, you can also **navigate and read line by line** using Previous and Next Thumb Keys!