


BrailleNote Touch Plus 2.8 – Using the Spellchecker

1




To spell check your document, press **SPACE + DOTS 1+6** (the contraction 'ch'). A spell-checking window will pop up.

OR

Long press the square **context menu button**, then navigate to **Editing Functions** and select it. Navigate to **Spell check** and select it.

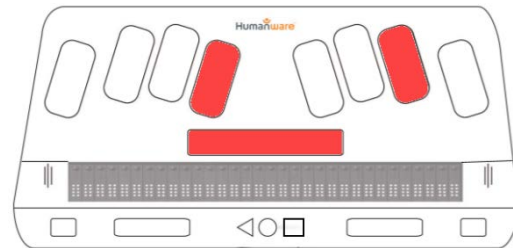


OR

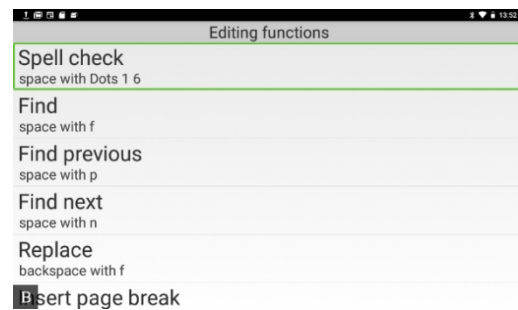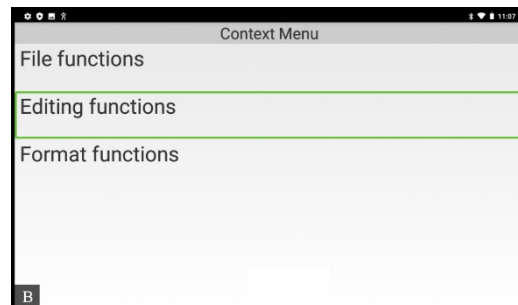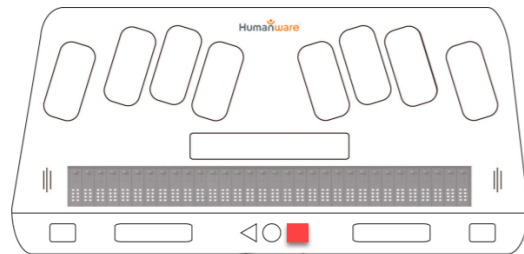


2

In the top of the spell-checker box, the **target word** is displayed in context; you can navigate to it to hear it.

There are various options for correcting the word. To **manually change the spelling**, navigate to the Misspelled Word Edit Box and press **enter** or a **cursor routing key**. You may now **delete** the word using Backspace, and then spell the word again. Press **Enter**. A pop-up window will be displayed asking if you want to “Correct Once” or “Apply to All;” navigate to the one you want and select it.

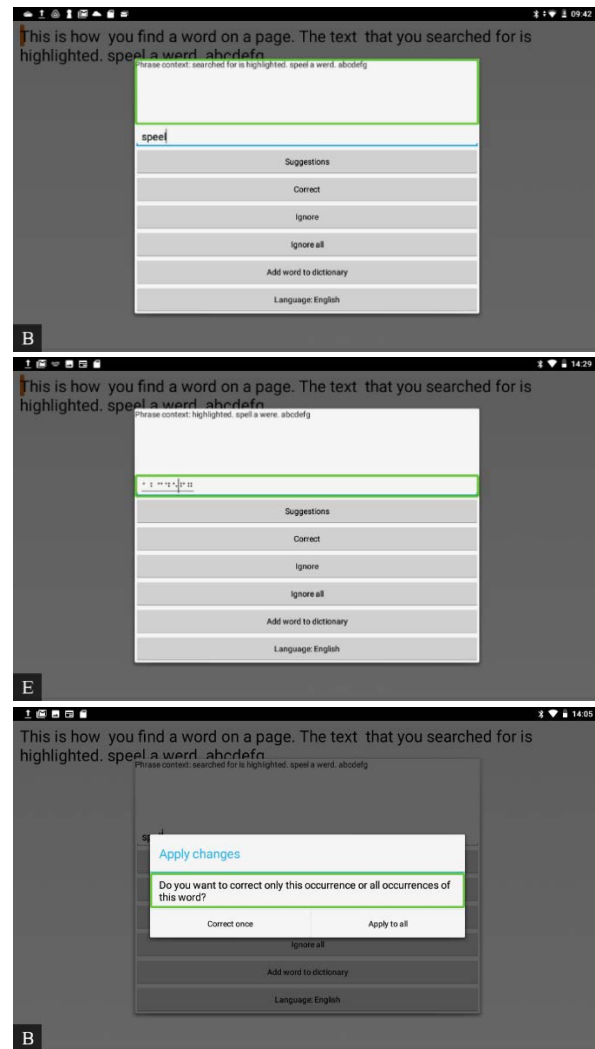

<p>3</p>	<p>Another option is to navigate to the first menu option, “Suggestions,” and select it (or type “S” in the Misspelled Word Edit Box.) Navigate through the list of words until you find the correct one and select it. You will again be asked to “Correct Once” or “Apply to All.”</p> <p>Shortcuts for navigating suggestions:</p> <ul style="list-style-type: none"> • Next=Space + Dot 4 • Previous=Space + Dot 1 	
<p>4</p>	<p>If you do not want to change the spelling of a selected word, navigate through the menu to “Ignore” (or type the letter “I” in the Misspelled Word Edit Box) and select it.</p>	
<p>5</p>	<p>When the spell checking is complete the pop-up will notify you that all corrections have been applied. Navigate to OK and select it.</p>	

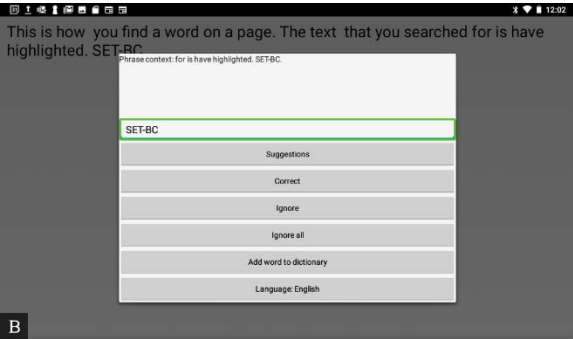6	<p>If you have a word in your document that you would like to add to your personal dictionary, navigate to “Add word to dictionary” when the word is selected and press enter/cursor routing key.</p> <p>You will receive a notification saying, ‘word added to dictionary.’”</p>	
---	--	--

Tip...

Download the HWBuddy App for support with your device.