

# Magnilink Keyboard Commands

| Function   | Mac                   | PC                |
|--|-----------------------|-------------------|
| <b>Natural Colours</b>   | fn + F1               | F1                |
| <b>Positive Picture with Increased Contrast</b>                                | fn + F2               | F2                |
| <b>Negative Picture with Increased Contrast</b>                                | fn + F3               | F3                |
| <b>Take a Snapshot</b>   | fn + F5               | F5                |
| <b>Take a Snapshot - Fast</b> (saves to last folder or Magnilink folder)       | fn + Shift + F5       | Shift + F5        |
| <b>Start/Stop Video Recording</b>  | fn + Alt + F5         | Ctrl + F5         |
| <b>Start Video Recording - Fast</b> (saves to last folder or Magnilink folder) | fn + Alt + Shift + F5 | Ctrl + Shift + F5 |
| <b>View Stored Snapshots</b>   | fn + F6               | F6                |

|   |                       |                    |
|---|-----------------------|--------------------|
| <b>View Last Stored Snapshot - Fast</b> | fn + Shift + F6       | Shift + F6         |
| <b>View Stored Videos</b>               | fn + Alt + F6         | Ctrl + F6          |
| <b>View Last Stored Video - Fast</b>    | fn + Alt + Shift + F6 | Shift + Ctrl + F6  |
| <b>Preset Split Modes</b>               | fn + F8               | F8                 |
| <b>Information and Setting Window</b>   | fn + Alt + F9         | F9                 |
| <b>Return to Default Settings</b>       | fn + Alt + F11        | F11                |
| <b>Zoom In</b>                          | Arrow Up              | Arrow Up           |
| <b>Zoom Out</b>                         | Arrow Down            | Arrow Down         |
| <b>Overview</b>                         | Alt + O               | Ctrl + O           |
| <b>Decrease Brightness</b>              | Arrow Left            | Arrow Left         |
| <b>Increase Brightness</b>              | Arrow Right           | Arrow Right        |
| <b>Guiding Line Left/Up</b>             | Alt + Arrow Left      | Ctrl + Arrow Left  |
| <b>Guiding Line Left/Up</b>             | Alt + Arrow Up        | Ctrl + Arrow Up    |
| <b>Guiding Line Right/Down</b>          | Alt + Arrow Right     | Ctrl + Arrow Right |

|   |                  |                   |
|---|------------------|-------------------|
| <b>Guiding Line Right/Down</b>                    | Alt + Arrow Down | Ctrl + Arrow Down |
| <b>Toggle "always on top"</b>                     | Alt + A          | Ctrl + A          |
| <b>Minimize Magnilink</b>                         | Cmd + M          | Ctrl + M          |
| <b>Rotate the Camera Picture Clockwise</b>        | Alt + R          | Ctrl+ R           |
| <b>Rotate the Camera Picture Counterclockwise</b> | Shift + Alt + R  | Shift + Ctrl + R  |
| <b>Mirror Camera Image</b>                        | Alt + I          | Ctrl + I          |
| <b>Freeze Camera Image</b>                        | Alt + F          | F                 |
| <b>Autofocus On/Off</b>                           | Alt + J          | Ctrl + J          |
| <b>Open the Preferences Dialog</b>                | Cmd + ,          |                   |
| <b>Open the Manual</b>                            | Cmd + ?          |                   |
| <b>Global Short Cut Commands</b>                  |                  | Ctrl + Alt + S    |
| <b>OCR-Process</b>                                | fn + F4          | F4                |

|   |               |                  |
|---|---------------|------------------|
| <b>Process with OCR and Add Text</b>            | fn + Alt + F4 | Ctrl + F4        |
| <b>Toggle Column Selector</b>                   | Alt + S       | Ctrl + S         |
| <b>Copy Text</b>                                | Cmd + C       | Ctrl + C         |
| <b>Paste Text</b>                               | Cmd + V       | Ctrl + V         |
| <b>Paste and Add Text</b>                       | Cmd + Alt + V | Ctrl + Shift + V |
| <b>Open Text Document</b>                       |               | Alt + F6         |
| <b>Switch Between Camera View and Text View</b> | V             | V                |
| <b>Display Mode</b>                             | B             | B                |
| <b>Start/Pause Reading</b>                      | 1             | 1                |
| <b>Stop Reading</b>                             | 2             | 2                |
| <b>Read Previous</b>                            | 3             | 3                |
| <b>Read Next</b>                                | 4             | 4                |
| <b>Switch Reading Mode</b>                      | 5             | 5                |
| <b>Slower</b>                                   | 6             | 6                |

|                        |   |   |
|------------------------|---|---|
| <b>Faster</b>          | 7 | 7 |
| <b>Reduce Volume</b>   | 8 | 8 |
| <b>Increase Volume</b> | 9 | 9 |
| <b>Next Voice</b>      | 0 | 0 |