


How to take a Rehadapt mount off and on

1 Remove the device.

2 Next, remove the top horizontal and vertical tubes at the black swivel clamp.

3 At the base coupler, open the red lever and pull out the pin.

4 Gently unweight the tube with one hand and pull straight out with the other hand. There is a magnet, so some resistance is normal.

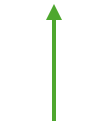

Re-connect the device. Check that the safety pin securely locks in place.

Re-insert the top part of the mount.

To re-assemble, support the weight of the tube. Line up the base coupler and let the magnet pull to connect. Do not force or wiggle.

Insert the pin and close the red lever.



Maintenance and Care of a Mount



Check bolts on a regular basis and tighten as needed. Wipe mount with a damp cloth occasionally.



Equipment Only Support (EOS) is available for two years after the service year.



Always check with the vendor before using cleaning agents or lubricants.



School teams can notify school therapist and Loan Bank if there are concerns with a mount.